**Biosigma SpA**

**a Dominique Dutscher Company**

Via Valletta, 6 | 30010 Cantarana di Cona (VE), Italy | Tel. ++39 0426 302224 (r.a.)

| Fax ++39 0426 302228 | SMS ++39 348 4077376 |

E-mail info@biosigmaeu.com | http://www.biosigma.com |

**M521D**

Rev. 29/09/2020

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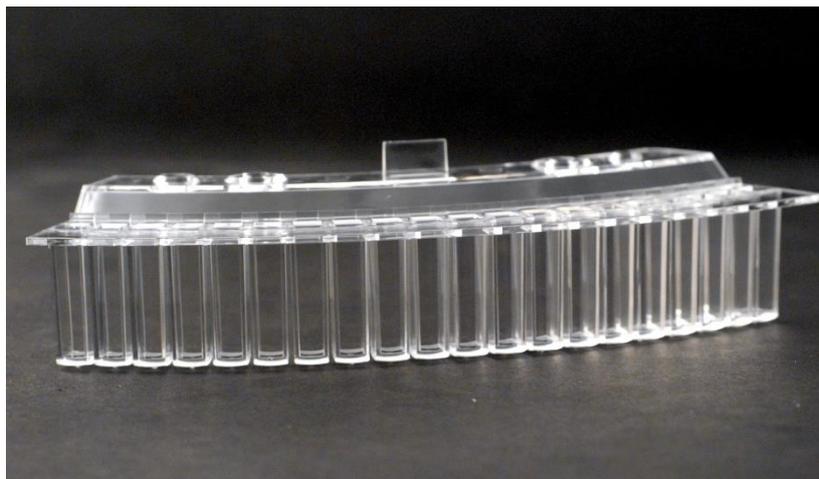
**Cuvette Usable on Hitachi 717/914 in Methacrylate**

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**WARNING - CHANGE CUVETTES:** it is recommended to soak in advance cuvettes in a 2% HITERGENT solution for some hours (preferably overnight). RINSE plenty with demineralised water. Then insert the cuvettes on the reaction plate, only when the incubation bath is 37°C, and perform a cell blank. The reaction cuvettes may break if left to dry out after use. If not used for any given length of time, store completely immersed in a HITERGENT solution, diluted 50 times (2%). Ensure that the reaction bath with water does not dry out. As specified in the Hitachi manuals, the cuvettes can be used for up to a month if the above instructions are carefully followed.

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**DESTINATION USE:**

Disposable labware intended to be used on laboratory to analyze biological samples on clinical chemistry analyzers optical reading.

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*The use and the manipulation of the devices **are reserved to the medical and technical staff** with the necessary professional qualification.*

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### **MATERIALS:**

*The materials used for the production of the in vitro labware devices are selected on the base of the properties indicated for the individual destination use.*

*The device BSA096 is manufactured in METHACRYLATE.*

***Methacrylate** is already known as raw material for manufacturing of in vitro medical device. Its main characteristics are: rigid, transparent, resistant to atmospheric agents, it replace the glass in every its application in which it reach high temperature (lower than 90-100 °C).*

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### IN VITRO DIAGNOSTIC MEDICAL DEVICE for professional use only

- Use only for laboratory analysis
- Use gloves: contamination risk if contents leaks
- Keep out of flame or heat sources which might damage the product
- Do not use after expiry date or if packing is opened
- Do not vary the intended purpose of the product
- Keep out of reach of children
- Store in dry place, temperature range: min. - 10 °C max 50°C
- Disposal: use appropriate personal protective equipment and act according to applicable regulations

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CLASSIFICATION

In order to the Directive 98/79/CE concerning medical-diagnostic in vitro devices receipt in Italy with D.L. n. 332 G.U.R.I. n. 269 dated 17 November 2000, Biosigma SpA, defines these devices as medical-diagnostic in vitro devices and it classifies as "Other medical-diagnostic in vitro devices", in order to make conformable the attachment III of the directive.

The Declaration of Conformity in accordance to the Directive 98/79/CE is available, on request, to the Quality Department Biosigma SpA

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BIOSIGMA SpA