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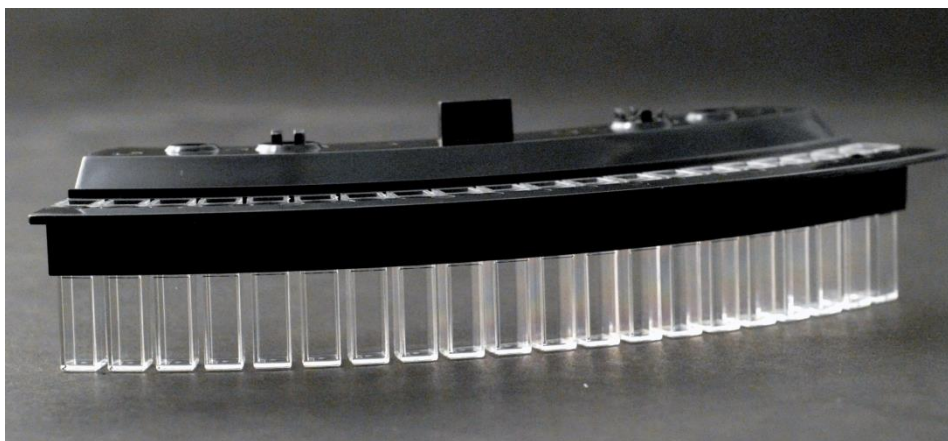
WARNING - CHANGE CUVETTES: it is recommended to soak in advance cuvettes in a 2% HITERGENT solution for some hours (preferably overnight). RINSE plenty with demineralised water. Then insert the cuvettes on the reaction plate, only when the incubation bath is 37°C, and perform a cell blank. The reaction cuvettes may break if left to dry out after use. If not used for any given length of time, store completely immersed in a HITERGENT solution, diluted 50 times (2%). Ensure that the reaction bath with water does not dry out. As specified in the Hitachi manuals, the cuvettes can be used for up to a month if the above instructions are carefully followed.

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DESTINATION USE:

Disposable labware intended to be used on laboratory to analyze biological samples on clinical chemistry analyzers optical reading.

FOR IN VITRO USE ONLY.

The use and the manipulation of the devices **are reserved to the medical and technical staff** with the necessary professional qualification.

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MATERIALS:

The materials used for the production of the in vitro labware devices are selected on the base of the properties indicated for the individual destination use.

The device BSA098 is manufactured in Bionex.

***Bionex** is a polymer with an high grade of purity, great chemical resistance, low fluorescence and optical stability.*

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IN VITRO DIAGNOSTIC MEDICAL DEVICE

for professional use only

- *Use only for laboratory analysis*
- *Use gloves: contamination risk if contents leaks*
- *Keep out of flame or heat sources which might damage the product*
- *Do not use after expiry date or if packing is opened*
- *Do not vary the intended purpose of the product*
- *Keep out of reach of children*
- *Store in dry place, temperature range: min. - 10 °C max 50°C*
- *Disposal: use appropriate personal protective equipment and act according to applicable regulations*

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CLASSIFICATION

In order to the Directive 98/79/CE concerning medical-diagnostic in vitro devices receipt in Italy with D.L. n. 332 G.U.R.I. n. 269 dated 17 November 2000, Biosigma SpA, defines these devices as medical-diagnostic in vitro devices and it classifies as "Other medical-diagnostic in vitro devices", in order to make conformable the attachment III of the directive.

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The Declaration of Conformity in accordance to the **Directive 98/79/CE** is available, on request, to the Quality Department Biosigma SpA

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