Biosigma S.p.A.

a Dominique Dutscher Company

Via Valletta, 6 | 30010 Cantarana di Cona (VE), Italy | Tel. ++39 0426 302224 (r.a.) |
Fax ++39 0426 302228 | E-mail info@biosigmaeu.com | http://www.biosigma.com |

M521D

Rev. 13/04/2021

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- ▶ *Material: Plastic material 100 % virgin Polypropylene*
- ▶ *Colour: Yellow body and red lid in order to signal the dangerous content
Materials coloured with masterbatch without heavy metals and cadmium Materials compliant to REACH*
- ▶ *Thickness: 1,8 mm bottom and side walls - 1.8 mm cover and closing system*
- ▶ *Shape: truncated cone*
- ▶ *Weight: 87gr.*
- ▶ *Max load: 400 gr.*
- ▶ *Nominal capacity: 0.600 Lt.*
- ▶ *External dimensions: Ø100x145 H mm*

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Press strongly around the lid with the palms of the hands.. A click will be heard at each pressure, showing the closure of the lid.

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▶ Turn the movable part of the lid clockwise, opening it according to the size of the object to dispose. Close the container permanently.

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The container can be closed temporarily using the cap with the concave side upwards. In this position, reopening is possible

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The container can be permanently closed using the cap with the concave side down until click. In this position, the closure is definitive

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▶ *During use, open and close the container using the cap on the side of the temporary closure*

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▶ *Close the container permanently using the cap from the side of the final closure*

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Once filled, the containers have to be disposed through incineration, releasing no toxic emissions.

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Use the "V" removal device to separate needle and syringe.

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Use the "marguerite" device to unscrew the insulin pen.

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Table tray with area set up to fix a container of PBS 0.6 Lt.

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Table support with suction cups to fix the 0.6 Lt. PBS container on the work surface

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- ▶ *Use the product only with the appropriate lid*
- ▶ *Do not use the product before fitting the lid*
- ▶ *Do not attempt to rescue objects already in the container*
- ▶ *Respect the filling level indicated: 80% of real capacity*
- ▶ *Do not try to open the container once permanently and irreversibly sealed*
- ▶ *Container to dispose according to the existing law*
- ▶ *Follow the instructions of use to ensure the safety of the container*
- ▶ *Use handles to move the container*

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- ▶ Closure system integral with the lid, device that prevents the contents from leaking or overturning
- ▶ Easy to use, easy to handle, practical
- ▶ Resistance to perforation of pungent and sharp objects normally used
- ▶ Provided with a handle
- ▶ Provided with a shaped hole for detaching needles
- ▶ Autoclavable (even after this process it retains the NFX 30-500 characteristics)
- ▶ Indication of maximum filling level
- ▶ Permanent irreversible closure
- ▶ Availability of various supports
- ▶ Disposal by incineration without emission of harmful gases