**Biosigma S.p.A.**

**a Dominique Dutscher Company**

Via Valletta, 6 | 30010 Cantarana di Cona (VE), Italy | Tel. ++39 0426 302224 (r.a.) |  
Fax ++39 0426 302228 E-mail [info@biosigmaeu.com](mailto:info@biosigmaeu.com) | <http://www.biosigma.com> |

**M521D**

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**SCHEDA TECNICA / TECHNICAL DATA SHEET**

**REF: BSC740** **CE**

**FERTILITY COLLECTOR**

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**Kit for seminal fluid collection - STERILE**

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**IN VITRO DIAGNOSTIC MEDICAL DEVICE**  
**Intended exclusively for professional use**

RECOMMENDATIONS FOR USE (purely  
informative)

- Use only for laboratory analysis
- In case of spillage of the content use gloves danger of contamination
- Do not use expired product
- Do not change the intended use
- Not suitable for children
- Store in a dry place, away from sources of heat and from direct sunlight
- Disposal: use the appropriate PPE and dispose of according to local regulations

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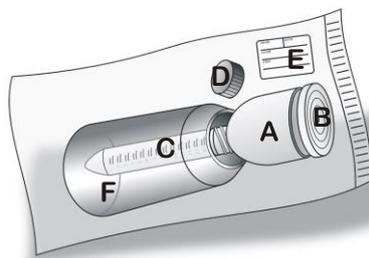
### INTENDED USE

*The KIT FOR FERTILITY are disposable devices used for the collection of sputum sample from the patient to the laboratory and use for analysis in vitro reserved for the medical and technical personnel with the necessary qualifications and professional preparation.*  
**USE ONLY IN VITRO.**

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### KIT CONTENTS:

- Funnel. (A)
- Plug for funnel (B).
- Test tubes, graduated, vol. 15ml. (C)
- Screw cap for tube. (D)
- Label for test tube. (E)
- Support for tube. (F)
- Instructions for use

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### INSTRUCTIONS FOR USE (To the patient)

- *Open the sterile bag and make sure that the funnel is inserted into the tube of sample collection.*
- *expectorate inside the funnel and then plug the funnel itself.*
- *Wait until all the sputum has entered into the tube leaving it in the holder.*
- *Separate the tube from the funnel and close it with the cap. The funnel can be thrown away.*
- *The tube should be stored at room temperature (15-30 ° C). Deliver the test tube containing the sample tightly closed.*

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### **RAW MATERIALS**

*The KIT FOR FERTILITY are produced in various materials such as polystyrene (PS) (TUBE), polypropylene (PP) (SUPPORT FOR FUNNEL), Polyethylene (PE) (FUNNEL AND RELATING CAP).*

**Polystyrene** is a rigid polymer, non-toxic, with excellent dimensional stability and good resistance to aqueous solutions but limited resistance to solvents. It's perfectly transparent and is commonly used for disposable products from the laboratory. At room temperature the products are rigid and therefore can crack or break if they fall from a laboratory bench or higher heights.

**Polypropylene** is a translucent material, lightweight, resistant to autoclaving. It has excellent chemical resistance (at room temperature is not dissolved by any solvent).

**Polyethylene** is a thermoplastic polymer, slightly opaque, lightweight, unbreakable, excellent chemical resistance, good resistance to solvents. Particularly used for the production of Pasteur pipettes and plugs for test tubes.

### **MATERIALS USED FOR PACKAGING PRODUCTS**

All components used for packaging are produced in non-polluting materials, suitable for the purpose for which they are intended.

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*For sterile products, available on request statements of beta treatment for Lot in possession. TREATMENT VALIDATED. All products sold as "STERILE" are made in accordance with UNI EN 556 and EN 11135.*

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*The Declaration of compliance with Directive 98/79 / EC on in vitro diagnostic medical devices is available on request from the Office Quality Biosigma S.p.A.*