Biosigma S.p.A.

a Dominique Dutscher Company

Via Valletta, 6 | 30010 Cantarana di Cona (VE), Italy | Tel. ++39 0426 302224 (r.a.) |

Fax ++39 0426 302228 |

E-mail info@biosigmaeu.com | <http://www.biosigma.com> |

M521D

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06/04/2021

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DISPOSABLE ENDOCERVICAL SAMPLING BRUSH NOT STERILE

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CE Marked product - manufactured in compliance with 93/42/EEC Directive Annex VII

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DIRECTIONS FOR USE

The device is intended for the execution of endocervical canal cytology smears.

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PTHALATES: the material production of the devices does not contain phthalates.

LATEX FREE: The production material and primary and secondary packaging are latex-free

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PACKAGING

The product is packaged in cellophane bags. Such a packaging ensures a 'proper waterproof protection and allows the product to reach the final consumer in safe hygienic conditions.

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STORAGE

Store in a dry place. Do not expose to direct sunlight or heat sources.

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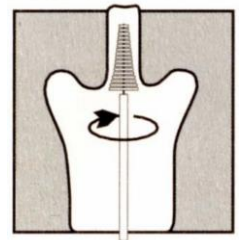
INSTRUCTIONS FOR USE

Carry out the sampling by inserting the device gently in the endocervix, for the complete length of the brush head.

Rotate the bristles by 360° in one manoeuvre only and then take it out.

Deposit the sampled material on a slide, by rotating and pushing gently the brush head on the slide.

Fix immediately by a cytological fixing spray and then dispose of the device.

Caution: in case of pregnancy use it gently. Do not use it for endometrial sampling.

WARNINGS :

- The device must be used by physicians or by trained staff only to carry out endocervical samplings.

- The product is disposable , therefore do not re-use it. Dispose after use.

DISCHARGE AFTER USE

The product is disposable and therefore not re-usable. It can be discharged or burnt with other garbage in accordance with the local authority rules. Burning does not cause any toxic residues.