Biosigma S.p.A.

a Dominique Dutscher Company

Via Valletta, 6 | 30010 Cantarana di Cona (VE), Italy | Tel. ++39 0426 302224 (r.a.) |
Fax ++39 0426 302228 |

E-mail info@biosigmaeu.com | <http://www.biosigma.com> | www.biosigma.it

M521D

Rev. 30/03/2021

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DESCRIPTION

A complete range of Devices ready to use, composed by a foam swab and a test tube with screw cap containing a liquid medium, suitable for the transportation and preservation of the clinical microbiological samples. The Devices standard dimensions make them suitable both for use with automatic instruments whether for manual use. The liquid medium type allows to dilute the samples easily and rapidly obtaining a homogeneous solution ready to be used and appropriate in order to obtain many samples equal rates to carry out different analysis.

Liquid Transport Swabs are packed in peel-pack, easy opening, totally in plastic material for the Devices optimal protection. The Devices are produced according to GMP (Good Manufacturing Practice), eventually in accordance with CLSI M40-A (Quality Control of Microbiology Specimen Transport Devices), classified and certified as Medical Devices Class II sterile (invasive surgery for temporary use).

Liquid Transport Swabs are sterile (SAL 10⁻⁶), Pyrogen free & Latex free.

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CE 0426 MD marked sterile Swab - manufactured in compliance with 93/42/EC Directive (Italian D.lgs 46 del 24/02/1997) + 2007/47/EC - Class IIa sterile
CE IVD marked sterile Test Tube with medium - manufactured in compliance with 98/79/EC Directive (Italian D.lgs 332 del 08/09/2000)

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INTENDED PURPOSE

Swab Intended purpose is "MEDICAL DEVICE" CLASS IIA STERILE. Swabs are sterile and ready-for-use systems intended for clinical samples drawing, transport and maintenance for cultural exam. The swab is suitable for short contact with the human body including surgical wounds. Test Tubes are classified as IVD and are intended only for the preservation of samples. Product must be used as directed, Microbiotech is not responsible for any unauthorized use.

For professional use only.

National classification of medical devices (CND - For Italian law) > V901302 (Transport swabs for collection sample)

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APPLICATOR

Special polyurethane foam, medical grade, swab tip that allows a better clinical sample collection and the following immediate release into the liquid transport medium. The soft foam bud is more comfortable for patients, and has significant advantages for both conventional and molecular methods. It

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has a plastic shaft with "breakpoint" to allow the swab breaking into the test tube and the successive collection by the screw "capture" cap after the tube closing. It allows to dispose of the swab together with the cap in a safer way.

Available in two versions on sampling site base:

- **Standard tip:** swabs, standard dimensions, for buccal, nasal, throat, vagina, rectal, cutaneous and wound sampling;
- **Fine tip:** swabs, small dimensions, for paediatric, urethral, urogenital and nasopharyngeal sampling.

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STANDARD TIP**PUNTA STANDARD**

This specialty applicator is tipped with medical grade polyurethane foam. It is bonded securely to the plastic handle. The best choice for buccal swab, for general specimen collection for testing and screening. Used in Pediatrics, Ophthalmology, Veterinary care, and oropharyngeal applications.

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Handle / Asta Ø 2.54 mm (0.1")
Length / Lunghezza 149.225 mm (5.875")

FINE TIP**PUNTA FINE**

The special small applicator is tipped with medical grade polyurethane foam. It is bonded securely to the plastic handle. The best choice for paediatric, urethral and nasopharyngeal applications.

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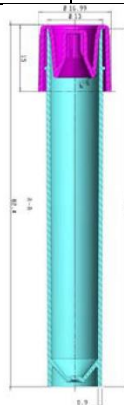
TEST TUBE

Liquid Transport Medium is contained in a cylindrical test tube Ø 13x80 mm, in transparent, unbreakable medical polypropylene (PP), to preserve sample and users safety. With screw cap in polyethylene (HDPE) with "capture cap" technology, The specific color cap identifies instantly and secure the medium content. Pre-labelled on test tube to allow a correct patient and sample identification. Item code, lot number and expiry date printed on each label for a full traceability.

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Rev. 30/03/2021

SCHEMA TECNICA PRODOTTO / TECHNICAL DATA SHEET**AMIES LIQUID TRANSPORT MEDIUM**

Swabs with pink screw cap test tube containing 1 ml of liquid AMIES transport medium for the sampling and preservation of aerobes, anaerobes and other fastidious bacteria such as *Streptococcus pyogenes*, *Neisseria gonorrhoeae*, *Bacteroides fragilis*, *Haemophilus influenzae*, *Streptococcus pneumoniae*, etc within 48 hours (24 hours for *Neisseria gonorrhoeae*) at room or refrigerated temperature. They can be used for the collection and preservation of mycobacterium and fungi. Amies is a medium of conservation consisting of inorganic phosphate, calcium and magnesium salts and sodium chloride with a reduced environment due to the presence of sodium tioglicollato.

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Swabs with blue screw cap test tube containing 1 ml of liquid STUART transport medium for the sampling and preservation of bacteria *Neisseria gonorrhoeae*, *Trichomonas vaginalis*, *Haemophilus influenzae*, *Bacteroides fragilis*, *Pseudomonas aeruginosa*, *Streptococcus pneumoniae*, *Streptococcus pyogenes*, *Corynebacterium diphtheria*, enterobacteria and bacteria of the upper respiratory tract. These microorganisms are kept viable for no more than 24 hours. More resistant bacteria are kept for 72 hours; rule of thumb is, however,

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to submit to the microbiological analysis, as soon as possible, all samples.

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Streptococcus pyogenes (ATCC® 19615): good growth at 35°C / buona crescita a 35°C

CARY BLAIR LIQUID TRANSPORT MEDIUM

Swabs with green screw cap test tube containing 2 ml of liquid CARY BLAIR transport medium, for the fecal or rectal clinical samples and for preservation of enteric pathogens agents as *Salmonella*, *Vibrio*, *Escherichia coli*, *Shigella flexneri*, *S. typhimurium*, *E. faecalis* (VRE), *V. parahaemolyticus*, *Y. enterocolitica*, *C. jejuni*, *C. difficile*, *Campylobacter*. The Cary Blair Transport Liquid Medium is especially suitable in epidemiological investigations for *Vibrio parahaemolyticus*, rectal swabs, especially when they have to be transported to a central diagnostic laboratory.

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Swabs with white screw cap test tube containing 2 ml of liquid medium SELENITE (Selenite broth) type for the fecal clinical samples or rectal swab, designed for the transportation, survival and isolation of *Salmonella enterica* and *Shigella sonnei*, reducing the growth of the other enteric pathogens like *Escherichia coli* and fecal streptococci. The selenite present in the medium inhibit microorganisms other than *Salmonella*, particularly coliforms and enterococci. *Pseudomonas* and *Proteus* are not inhibited. Dipotassium phosphate maintains constant pH and reduces the toxicity of selenium.

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Swabs with light blue screw cap test tube containing 2 ml of sterile buffered saline sodium chloride solution, pH 7.2. Particularly suitable to obtain immediately a ready to use dilute clinical sample solution for inoculation and analysis.

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*Staphylococcus aureus (ATCC® 25923): good growth at 37°C / buona crescita a 37°C***RESULTS**

Bacteria survival in the transport media depends upon a series of factors such as bacteria type, transport time, preservation temperature, bacteria concentration in the sample and transport medium formula. Devices ensure the survival of several micro-organisms for 24-48 hours. In the case of fastidious bacteria such as Neisseria Gonorrhoeae and Streptococcus pneumoniae, the samples on the swab shall directly be put on a plate with culture medium or immediately transported to the laboratory and cultivated within 24 hours. Use of the Liquid Transport Swabs in conjunction with rapid diagnostic kits and instruments must be validated prior to use by the user.

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PRECAUTIONAL MEASURES

- The device must be suitable for both the selected transport medium (if present) and the defined sampling site.
- Use only for the collection and transport of human biological samples for the detection of potential pathogens such as aerobic, anaerobic and facultative anaerobic bacteria, compatible with the type of device selected.
- Do not deviate from the intended use.
- Do not use the product if it is expired or the package is opened/damaged. Sterility guaranteed if unopened.
- Use the device following aseptic procedures.
- Single-use device; do not reuse. Reusing the device could contaminate the sample and/or the patient.
- Keep the device away from flames and heat sources.
- The fiber is only guaranteed to adhere to the shaft for instant sampling.
- The shaft is breakable, exert moderate pressure during sample collection.
- Store in a cool, dry place at a temperature between +2°C (35.6°F) and +30°C (86°F). Do not freeze.
- After use, the device may contain infectious microorganisms. Use appropriate PPE and dispose of the test tube and swab according to current regulations for medical waste.
- Keep out of flame or heat sources which might damage the product.
- Keep out of reach of children.
- Strictly follow the user's instructions.

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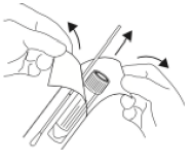
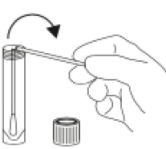
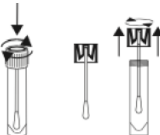
SPECIMEN CULTURES IN THE LABORATORY

Manual Processing

Vortex or mix well by shaking the test tube with the swab inside to release cells and create even suspension in the liquid medium. Remove the cap with swab applicator. Using the swab applicator, Streak the agar plate while rolling the swab tip. If additional plates are required replace swab back into the vial for a few seconds to recharge the swab. Using a pipette with a sterile pipet tip, transfer the suspension onto an agar plate.

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**Biosigma S.p.A.****a Dominique Dutscher Company**