Biosigma S.p.A.

a Dominique Dutscher Company

Via Valletta, 6 | 30010 Cantarana di Cona (VE), Italy | Tel. ++39 0426 302224 (r.a.) |

Fax ++39 0426 302228 |

E-mail info@biosigmaeu.com | http://www.biosigma.com | www.biosigma.it

M521D

Rev. 30/03/2021

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DESCRIPTION

Devices ready to use, composed by a foam swab with a plastic shaft and a test tube with screw cap containing a room temperature stable viral transport medium, suitable for the transportation and preservation of Viruses (such as Influenza H1N1), Chlamydia, Mycoplasma and Ureaplasma specimens.
The Devices standard dimensions make them suitable both for use with automatic instruments whether for manual use. The liquid medium type allows to dilute the samples easily and rapidly obtaining a homogeneous solution ready to be used and appropriate in order to obtain many samples equal rates to carry out different analysis.
Foam Virus Liquid Swabs are packed in peel-pack, easy opening for the Devices optimal protection. The Devices are produced according to GMP (Good Manufacturing Practice), classified and certified as Medical Devices. Foam Virus Liquid Swabs are sterile (SAL 10 -6), Pyrogen free & Latex free.

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CE MD marked sterile Swab - manufactured in compliance with 93/42/EC Directive (Italian D.lgs 46 del 24/02/1997) + 2007/47/EC
CE IVD marked sterile Test Tube with medium - manufactured in compliance with 98/79/EC Directive (Italian D.lgs 332 del 08/09/2000)

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INTENDED PURPOSE

Swab Intended purpose is "MEDICAL DEVICE" CLASS IIA STERILE.
Swabs are sterile and ready-for-use systems intended for clinical samples drawing, transport and maintenance for cultural exam. The swab is suitable for short contact with the human body including surgical wounds. Test Tubes are classified as IVD and are intended only for the preservation of samples. Product must be used as directed, Microbiotech is not responsible for any unauthorized use.
For professional use only.
National classification of medical devices (CND - For Italian law) > V901302 (Transport swabs for collection sample)

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APPLICATOR

Special polyurethane foam, medical grade, swab tip that allows a better clinical sample collection and the following

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immediate release into the liquid transport medium. The soft foam bud is more comfortable for patients, and has significant advantages for both conventional and molecular methods. It has a plastic shaft with "breakpoint" to allow the swab breaking into the test tube and the successive collection by the screw "capture" cap after the tube closing. It allows to dispose of the swab together with the cap in a safer way.

Available in two versions on sampling site base:

- Standard tip: swabs, standard dimensions, for buccal, nasal, throat, vagina, rectal, cutaneous and wound sampling;
• Fine tip: swabs, small dimensions, for paediatric, urethral, urogenital and nasopharyngeal sampling.

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STANDARD TIP

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This specialty applicator is tipped with medical grade polyurethane foam. It is bonded securely to the plastic handle. The best choice for buccal swab, for general specimen collection for testing and screening. Used in Pediatrics, Ophthalmology, Veterinary care, and oropharyngeal applications.

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FINE TIP

PUNTA FINE

The special small applicator is tipped with medical grade polyurethane foam. It is bonded securely to the plastic handle. The best choice for paediatric, urethral and nasopharyngeal applications.

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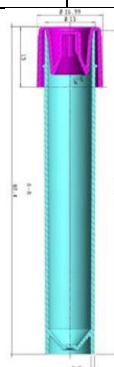
TEST TUBE

PROVETTA

Liquid Transport Medium is contained in a cylindrical test tube Ø 13x80 mm, in transparent, unbreakable medical polypropylene (PP), to preserve sample and users safety. With screw cap in polyethylene (HDPE) with "capture cap" technology, The specific color cap identifies instantly and secure the medium content. Pre-labelled on test tube to allow a correct patient and sample identification. Item code, lot number and expiry date printed on each label for a full traceability.

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VIRUS LIQUID TRANSPORT MEDIUM

Swabs with red screw cap test tube containing 1 or 3 ml of liquid VIRAL transport medium for the collection and transport of clinical samples containing viruses, chlamydiae, mycoplasmas and ureaplasmas from the collection site to the testing laboratory. The specimen transported in the Foam Virus Liquid Swabs can be used in the laboratory to perform viral, chlamydial, mycoplasmal and ureaplasma culture. Viral transport medium, stable at room temperature, consists of a balanced buffer solution to maintain neutral pH, antimicrobial agents and a preservative. Organisms viability maintained at room temperature and on ice. Suitable also for long term freeze storage. This product is not intended to be used for the collection and transport of general bacterial and fungal specimens.

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18 months / mesi

RESULTS

The survival and recovery of viruses, chlamydiae, mycoplasmas and ureaplasmas depends upon a series of factors such as infectious organisms type, transport time, preservation temperature, infectious organisms concentration in the sample. Specimens should be collected soon after the onset of symptoms when microorganism titers are at their highest. Specimens should be placed in the transport medium immediately following collection and promptly transferred to the laboratory for processing. For optimum recovery, specimens should be refrigerated during transport. For long term storage, specimens should be frozen at -70°C or colder. Use of the Foam Virus Liquid Swabs in conjunction with rapid diagnostic kits and instruments must be validated prior to use by the user.

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PRECAUTIONAL MEASURES

- The device must be suitable for both the selected

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transport medium (if present) and the defined sampling site;

- Use only for the collection and transport of human biological samples for the detection of potential pathogens such as viruses, chlamydiae, mycoplasmas and ureaplasmas, compatible with the type of device selected;
Do not deviate from the intended use;
Do not use the product if it is expired or the package is opened/damaged. Sterility guaranteed if unopened;
Use the device following aseptic procedures;
Single-use device; do not reuse. Reusing the device could contaminate the sample and/or the patient;
Keep the device away from flames and heat sources;
The fiber is only guaranteed to adhere to the shaft for instant sampling;
The shaft is breakable, exert moderate pressure during sample collection;
Store in a cool, dry place at a temperature between +2°C (35.6°F) and +25°C (77°F). Do not freeze;
After use, the device may contain infectious microorganisms. Use appropriate PPE and dispose of the test tube and swab according to current regulations for medical waste.
Keep out of flame or heat sources which might damage the product.
Keep out of reach of children.
Strictly follow the user's instructions.

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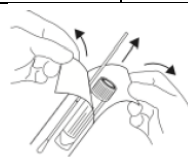
HOW TO USE

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
Open the blister pack from the "peel here" indication and aseptically remove the swab from the blister pack.

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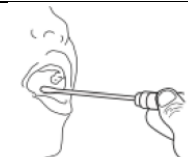Take the sample from the patient by applying moderate pressure.

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Unscrew the cap and insert the swab into the test tube, being careful not to spill the liquid medium contained inside.

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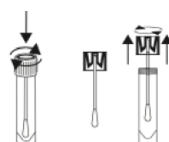
Break the swab in the test tube by placing the "breakpoint" indicated on the shaft of the swab against the edge of the test tube. Tilt it 180°, using moderate pressure (not applicable for aluminium shaft swabs). Discard the broken part of the shaft in accordance with current regulations for medical waste.

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Firmly tighten the cap onto the test tube and shake gently. Label the test tube with the patient's data and keep it at room temperature. Deliver it to the laboratory within 48 hours. When the cap is removed, the swab will also automatically be removed. Analyze the sample by performing standard clinical laboratory procedures.

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PACKING

Internal packing (pcs): 100
Dimensions: 340 x 225 x 175 mm
External packing (pcs): 600 pcs (6 x 100 pcs)
Dimensions: 540 X 360 X 460 mm

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