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VacuAptaca® - sterile vacuum test tubes manufactured in PET with cap and label yellow colour, with self-adhesive label applied on the test tube with conical cylindrical (chemical analysis of urine) bottom, without preservative. Cap in butyl rubber, easily pierceable and with repositionable safety cover cap splash proof and antiaerosol, with the function of operator protection. Inside the cover cap there is an easily pierceable pharmaceutical rubber cap. All materials used for the production are latex-free, heavy metal free and phthalates and sulphates free. The tubes are inside sterile with a SAL (Sterility Assurance Level) of 10⁻⁶ in accordance to UNI EN ISO 11137 and UNI EN ISO 11737 Rules. The tubes are classified as in Vitro Diagnostic Medical Devices according to Directive 98/79/CE and subsequent modifications and additions. Packaged in easy to handle racks of polystyrene foam sealed with polyethylene film.

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Collect urine sample in a collector with vacuum and sampling system with the cap tightly closed. Identify the sample with patient data

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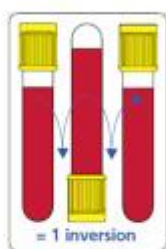
Remove the protective sticker from the cap and insert the tube into the adapter until the cap of the tube is not completely penetrated.

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
Wait until the tube has collected the correct urine volume, and then remove it from the container adapter. Identify the sample with patient data.

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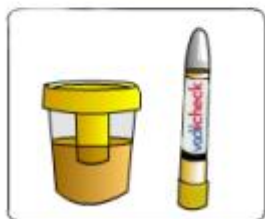In case of tube with preservative, reverse it gently for 5-8 times after sampling in order to obtain a homogeneous mixture of urine and a preservative.

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Re-apply the protective sticker on the container cap and dispose it and the container according to the regulations on force

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*Keep all the tubes to a temperature between +4 °C / +30 °C. Storage outside the up mentioned temperatures can cause a loss of efficiency of the tubes and therefore modify the results of the analysis test. Do not use the product beyond the expiration date. The maximum tolerance of the vacuum is $\pm 10\%$. The urine quantity taken with vacuum tubes can vary with **altitude**, temperature, atmospheric pressure, the test tube production date and sampling technique.*

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After collecting the last tube dispose all devices in special containers in accordance with the present regulations. All sampling devices must be considered potentially infected after use.

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Classificazione EDMA > 51091001 - Other containers for samples of human origin

Basic UDI-DI (Global Model Number – GMN) > 805577609FT15URINEVAC026U

In Vitro Diagnostic Medical Device "specimen receptacles" adapted to contain urine to perform in vitro diagnostic analysis laboratory. For professional use only.

National classification of medical devices (CND) > W05010202 (Urine collection tubes).

EDMA code > 51091001 - Other containers for samples of human origin

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- > To be used exclusively for the sampling and analysis of urine;
- > Do not change the intended use;
- > Tubes are internally sterile. Sterility guaranteed only to product integrity;
- > To be used only with compatible accessories (vacuum urine containers, holder, collection sets, etc.);
- > It is not recommended to transfer the urine sample from a syringe to the vacuum tube;
- > Do not use the product expired or damaged;
- > Use the device with aseptic procedures and using appropriate DPE (gloves, eye protectors, suitable clothing, etc.);
- > Do not reuse, disposable device;
- > Do not bring the device to flame or heat sources;
- > Keep in a cool and dry place, min temperature +4° C (39.2° F)/max +30° C (86° F). Do not expose to sunlight;
- > Contented additives are colorless or white/light yellow. Do not use the product if the color has varied;
- > Every device, after use, may contain infectious microorganisms, use the appropriate PPE and dispose specimen and sampling accessories according to the regulations on force.

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Step 1: Double arms holding. One hand pinches the bottom of the cap, and tweak it, the other hand pushes the cap upwards at the same time.

Step 2: Move the thumb before pull the cap out.

Step 3: Pull the cap upwards gently.

**How to close?**

Step 1: Make the cap right above the tube.

Step 2: Tweak and meanwhile press the cap to the tube.