Biosigma S.p.A.

a Dominique Dutscher Company

Via Valletta, 6 | 30010 Cantarana di Cona (VE), Italy | Tel. ++39 0426 302224 (r.a.)

| Fax ++39 0426 302228 |

E-mail info@biosigmaeu.com | <http://www.biosigma.com> |

M521D

Agg. del 17/12/2024

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The standardization of urinary sediment analysis can now be simpler and more accurate. It is important that the examination of the sediment is carried out always following the same procedures to guarantee the repeatability of the test, which is difficult to obtain with the traditional system currently used in many laboratories. Furthermore, the compaction of the urinary sediment creates an overlap of epithelial cells making it difficult to verify the presence of other cellular elements.

Advantages: The **STAN-TAP** allows rapid and repeatable preparation of the urinary sediment sample for microscopic analysis. Da utilizzare con / To be used with

BSP074 **Provetta conica in polistirolo
trasparente graduata a 1; 2,5; 5; 10 ml
Transparent polystyrene conical test
tube graduated to 1; 2.5; 5; 10 ml
Conf. 2000 pezzi/pcs**

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Fill the tube with the urine up to the level indicated by the maximum graduation on the tube (10 ml).

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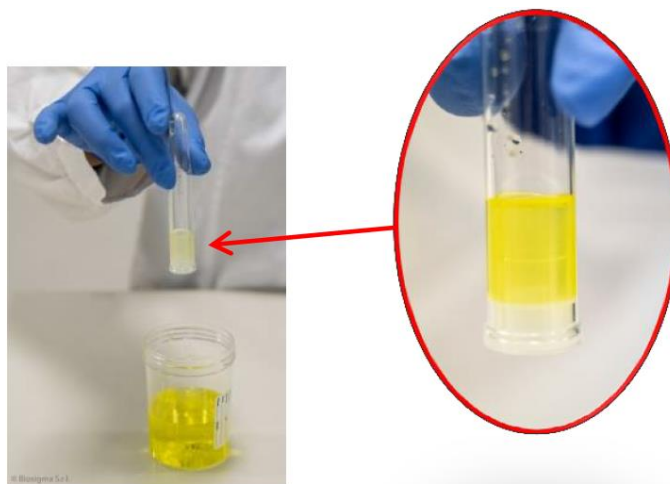
Insert the Stan-Tap by exerting slight pressure to allow it to fit into the tube.

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Decant excess urine by rapidly inverting the tube (by rotating 180°).

The sediment and urine residue up to 1 ml will remain in the tube and will be retained in the gap between the tube and the Stan-Tap.

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Bring the tube into a vertical position by rotating it 180°. Resuspend the sediment and proceed to the analysis.

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The STAN-TAP is a supplementary device to FAST-READ®

Compatible devices with STAN-TAP (BVS172):

REF	Descrizione <i>Description</i>	Conf. di vendita <i>Sales package</i>	Immagine <i>Image</i>
BSP074	Provetta conica in polistirolo trasparente graduata a 1; 2,5; 5; 10 ml <i>Transparent polystyrene conical test tube graduated to 1; 2.5; 5; 10 ml</i>	2000 pezzi/pcs	
BSP075	Provetta conica da 10 ml in PS, graduata a 1; 2,5; 5; 10 ml con etichetta <i>Conical test tube 10 ml in PS, graduated to 1; 2.5; 5; 10 ml with label</i>	2000 pezzi/pcs	
BSP112	Provetta conica da 10 ml in PS, graduata a 1; 2,5; 5; 10 ml con etichetta e tappo a pressione bianco. <i>Conical test tube 10 ml in PS, graduated to 1; 2.5; 5; 10 ml with label and white pressure cap.</i>	1500 pezzi/pcs	
BSA064	Tappo in PE alettato per provette diametro 13 mm. <i>Stopper in PE with wings for 13 mm diameter test tubes.</i>	3000 pezzi/pcs	
BSV140	Pipetta Pasteur in PE FAST READ. Pasteur pipette in PE FASTREAD.	6000 pezzi/pcs	
BVS100	FAST READ® 102 1 conf=100 vetrini= 1000 determinazioni 1 conf = 100 slides = 1000 determinations		