Biosigma s.r.l.

a Dominique Dutscher Company

Via Valletta, 6 | 30010 Cantarana di Cona (VE), Italy | Tel. ++39 0426 302224 (r.a.) |

Fax ++39 0426 302228 | SMS ++39 348 4077376 |

E-mail info@biosigmaeu.com | http://www.biosigma.com | www.biosigma.it

M521D

Rev. 21/11/2017

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**DRY SWABS IN TEST TUBES
TAMPONI SECCHI IN PROVETTA**

DESCRIPTION / DESCRIZIONE

Dry swabs, available in a wide range of materials and sizes, for various applications. They are contained in a labelled cylindrical test tube $\varnothing 12 \times 150$ mm for a more easy samples protection, in unbreakable medical PP, to preserve sample and users safety. Supplied with label on test tube and cap for patient and sample identification and as a seal indicating that the product has not been previously used. Cap has same diameter than test tube. Can be packed in medical peel-pack or in bulk. All the Devices are produced according to GMP (Good Manufacturing Practice) and are classified and certified as Medical Devices Class II sterile (invasive surgery for temporary use). The swabs are sterile irradiated (SAL 10^{-6}), Pyrogen free.

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CE 0426 MD marked sterile - manufactured in compliance with 93/42/EC Directive (Italian D.lgs 46 del 24/02/1997) + 2007/47/EC - Class IIa sterile

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FABBRICANTE / MANUFACTURER

Microbiotech Srl

Operating Headquarter: Via Achille Tamborino, snc – zona P.I.P. – 73024 Maglie (Lecce) – Italy

Legal Headquarter: Via Cosimo di Palma, 9 - 73100 Lecce (Lecce) – Italy

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Swab Intended purpose is "MEDICAL DEVICE" CLASS IIA STERILE. Swabs are sterile and ready-for-use systems intended for clinical samples drawing for cultural exam. The swab is suitable for short contact with the human body including surgical wounds. Product must be used as directed, we accept no responsibility for any unauthorized use.

For professional use only.

National classification of medical devices (CND - For Italian law) > V901301 (Dry swabs for collection sample)

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The applicator shaft of the swab is mounted on the tube plug which serves as an optimal grip for an efficient swab handling. Available in different versions on sampling site base:

- **Aluminium shaft, cotton or rayon tip:** atoxic shaft, with cotton or rayon tip, ideal for arduous or delicate sampling. The narrow dimension of the swabs shaft (only 0.9 mm), the high flexibility and strength, the small fiber tip, make it particularly suitable for urogenital, urethral, ocular, nasopharyngeal and paediatric uses.
- **Plastic shaft, cotton or rayon tip:** made in anti-shock polystyrene, very flexible but easily breakable as needed, with cotton or rayon tip, atoxic. Particularly suitable for mouth, throat, vagina, urogenital apparatus, skin, wounds and surgical wounds.
- **Wooden shaft, cotton tip:** made in 100% cotton tip (European Pharmacopea) and atoxic wooden stick. Swabs for general purpose, suitable for for throat, vagine, skin and wounds drawings.

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TEST TUBE / PROVETTA

Swab is contained in a cylindrical test tube \varnothing 12 x 155 mm, in transparent, unbreakable medical polypropylene (PP), to preserve sample and user's safety. With pressure cap in polyethylene (HDPE) with specific colour according to stick type. Pre-labelled on test tube to allow a correct sample identification. Item code, lot number and expiry date printed on each label for a full traceability.

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PLASTIC STICK / ASTA PLASTICA

Sterile swabs plastic stick and cotton or rayon tip, in polypropylene \varnothing 12 x 150 mm test tube with label.

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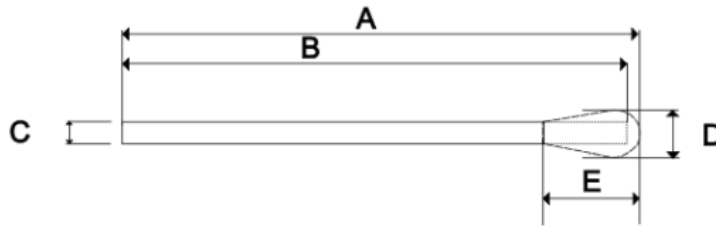
This specialty swab is made with cotton or rayon fibers. It is bonded securely to a rigid white birch polystyrene handle by an aqueous based adhesive.

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Absorbance / Assorbenza: 100 µL ± 10% in 10 sec. (acqua + TSB) / (water + TSB)

Dimensions / Dimensioni


A: 150 ±2 mm / B: 146 ±1 mm / C: Ø 2,5 ±0,1 mm / D: 5 – 6 mm / E: 15 mm ±2 mm

Table with 5 columns: Item code, Tipo / Type, Packaging, Manufacturer item Code, R.D.M. 1. Rows include BSM0324, BSM032, and SHELF LIFE / SCADENZA.

ALLUMINIUM STICK / ASTA ALLUMINIO

Sterile swabs aluminium stick and cotton tip, in polypropylene Ø 12 x 150 mm test tube with label.

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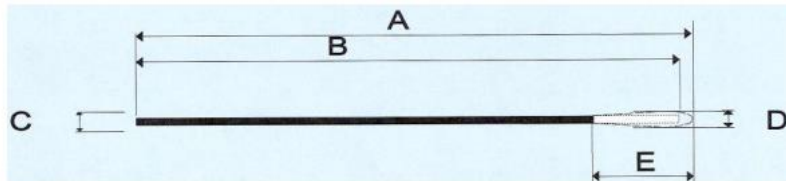Swabs with cotton tip, aluminium stick, atoxic, resistant to sudden changes of temperature. Atoxic glue to fix the tip. Very thin shaft and small tip for paediatric, urethral, urogenital and nasopharyngeal sampling. Without break point.

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Absorbance / Assorbenza: 20 µL ±10% in 10 sec. (acqua + TSB) / (water + TSB)

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- The device must be suitable for both the selected transport medium (if present) and the defined sampling site;
- Use only for the collection and transport of human biological samples for the detection of potential pathogens such as aerobic, anaerobic and facultative anaerobic bacteria, compatible with the type of device selected;
- Do not deviate from the intended use;
- Do not use the product if it is expired or the package is opened/damaged. Sterility guaranteed if unopened;
- Use the device following aseptic procedures;
- Single-use device; do not reuse. Reusing the device could contaminate the sample and/or the patient;
- Keep the device away from flames and heat sources;
- The fiber is only guaranteed to adhere to the shaft for instant sampling;
- The shaft is breakable, exert moderate pressure during sample collection;
- Store in a cool, dry place at a temperature between +2°C (35.6°F) and +30°C (86°F). Do not freeze;
- After use, the device may contain infectious microorganisms. Use appropriate PPE and dispose of the test tube and swab according to current regulations for medical waste.
- Keep out of flame or heat sources which might damage the product.
- Keep out of reach of children.
- Strictly follow the user's instructions.

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QUALITY ASSURANCE

All raw materials, components and every lot of finished product undergo strict quality inspections carried out by N.A. in order to check conformance to specifications.

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The content of the unopened and undamaged units is guaranteed to be sterile. Do not use the units in case of damage, dehydration or contamination. The Devices must be stored at +2°C (35.6°F) / +30°C (86°F) until used. Improper storage will result in a loss of efficacy. Do not use after expiration date indicated on label.

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If present, open the blister pack from the "peel here" indication. If the swab is in a test tube, remove the applicator from the test tube;

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Collect the sample from the concerned site;

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For a dry swab in a test tube, put the applicator into the tube, seal tightly with the cap and write the patient's data on the label.

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Manual Processing / Procedura manual

Remove the cap with swab applicator;

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Using the swab applicator, Streak the agar plate while rolling the swab tip.

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