Biosigma s.r.l.

a Dominique Dutscher Company

Via Valletta, 6 | 30010 Cantarana di Cona (VE), Italy | Tel. ++39 0426 302224 (r.a.) |

Fax ++39 0426 302228 | SMS ++39 348 4077376 |

E-mail info@biosigmaeu.com | http://www.biosigma.com | www.biosigma.it

M521D

Rev. 23/11/2017

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DESCRIPTION / DESCRIZIONE

A complete range of swabs for the collection and transport of bacteriological samples, with transport medium agar gel to grant the survival of specimens during their transportation to clinical laboratory. The transport swabs are composed of a soft rayon tip with plastic or aluminium applicator stick, atoxic and resistant to sudden changes of temperature, and a labelled cylindrical test tube Ø12x155 mm, in unbreakable medical PP, to preserve sample and users safety. Supplied with label on test tube and cap for patient and sample identification and as a seal indicating that the product has not been previously used. Cap has same diameter than test tube. All the Devices are produced according to GMP (Good Manufacturing Practice) and are classified and certified as Medical Devices Class IIa sterile (invasive surgery for temporary use). The swabs are sterile irradiated (SAL 10⁻⁶), in medical easy opening peel-pack, Pyrogen free.

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CE 0426 MD marked sterile Swab - manufactured in compliance with 93/42/EC Directive (Italian D.lgs 46 del 4/02/1997) + 2007/47/EC - Class IIa sterile

CE marked sterile Test Tube with medium - manufactured in compliance with 98/79/EC Directive (Italian D.lgs 332 del 08/09/2000)

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FABBRICANTE / MANUFACTURER

Microbiotech Srl

Operating Headquarter: Via Achille Tamborino, snc – zona P.I.P. – 73024 Maglie (Lecce) – Italy

Legal Headquarter: Via Cosimo di Palma, 9 - 73100 Lecce (Lecce) – Italy

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Swab Intended purpose is "MEDICAL DEVICE" CLASS IIA STERILE. Swabs are sterile and ready-for-use systems intended for clinical samples drawing, transport and maintenance for cultural exam. The swab is suitable for short contact with the human body including surgical wounds. Test Tubes are classified as IVD and are intended only for the preservation of samples. Product must be used as directed, Microbiotech is not responsible for any unauthorized use.

For professional use only.

National classification of medical devices (CND - For Italian law) > V901302 (Transport swabs for collection sample)

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APPLICATOR / TAMPONE

The applicator shaft of the swab is mounted on the tube plug which serves as an optimal grip for an efficient swab handling. Available in two versions on sampling site base:

- **Aluminium shaft, rayon tip:** atoxic shaft, with rayon tip, ideal for arduous or delicate sampling. The narrow dimension of the swabs shaft (only 0.9 mm), the high flexibility and strength, the small fiber tip, make it particularly suitable for urogenital, urethral, ocular, nasopharyngeal and paediatric uses.
- **Plastic shaft, Rayon tip:** made in anti-shock polystyrene, very flexible but easily breakable as needed, with rayon tip, atoxic. Particularly suitable for mouth, throat, vagina, urogenital apparatus, skin, wounds and surgical wounds.

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TEST TUBE / PROVETTA

Transport Medium is contained in a cylindrical test tube $\varnothing 12 \times 155$ mm, in transparent, unbreakable medical polypropylene (PP), to preserve sample and user's safety. With pressure cap in polyethylene (HDPE) with specific colour according to medium type. Pre-labelled on test tube to allow a correct sample identification. Item code, lot number and expiry date printed on each label for a full traceability.

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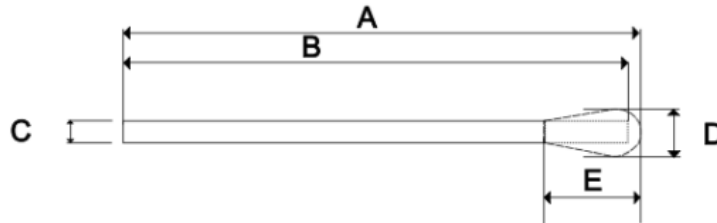
PLASTIC STICK, RAYON TIP / ASTA PLASTICA, PUNTALE IN VISCOSA

This specialty swab is made with rayon fiber. It is bonded securely to a rigid white birch polystyrene handle by an aqueous based adhesive.

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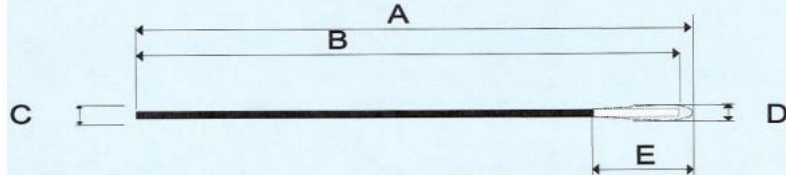
Absorbance / Assorbenza: 100 μ L \pm 10% in 10 sec. (acqua + TSB) / (water + TSB)

Dimensions / Dimensioni

A: 150 \pm 2 mm / B: 146 \pm 1 mm / C: \varnothing 2,5 \pm 0,1 mm / D: 5 – 6 mm / E: 15 mm \pm 2 mm

ALUMINUM STICK, RAYON TIP / ASTA ALLUMINIO, PUNTALE IN VISCOSA

Swabs with viscose (rayon) tip, aluminium stick, atoxic, resistant to sudden changes of temperature. Atoxic glue to fix viscose tip. Very thin shaft and small tip for paediatric, urethral, urogenital and nasopharyngeal sampling. Without break point.

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Absorbance / Assorbenza: 20 μ L \pm 10% in 10 sec. (acqua + TSB) / (water + TSB)

Dimensions / Dimensioni

A: 147 \pm 2 mm / B: 145 \pm 1 mm / C: \varnothing 0,9 \pm 0,1 mm / D: 2,2 \pm 0,5 mm / E: 12 \pm 2 mm

CLEAR AMIES / AMIES CHIARO

Sterile swabs with clear Amies agar gel medium. Amies transport medium allows the survival of many microorganisms, such as Shigella flexneri, Neisseria, Trichomonas vaginalis, Enterobacterias, Haemophilus, Corynebacterium, Streptococcus pneumoniae, Streptococcus pyogenes, Salmonella typhi, Brucella abortus.

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AMIES WITH CHARCOAL / AMIES CON CARBONE

Sterile swabs with Amies with charcoal agar gel medium. The Amies formula with charcoal has been shown to improve the recovery of *Neisseria gonorrhoeae*, *Vibrio cholerae* and is often moreover recommended for *Bordetella pertussis*.

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RESULTS / RISULTATI

Bacteria survival in the transport media depends upon a series of factors such as bacteria type, transport time, preservation temperature, bacteria concentration in the sample and transport medium formula.

Devices ensure the survival of several micro-organisms for 24-48 hours. In the case of fastidious bacteria such as Neisseria Gonorrhoeae and Streptococcus pneumoniae, the samples on the swab shall directly be put on a plate with culture medium or immediately transported to the laboratory and cultivated within 24 hours.

Use of the Agar Gel Transport Swabs in conjunction with rapid diagnostic kits and instruments must be validated prior to use by the user.

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- The device must be suitable for both the selected transport medium (if present) and the defined sampling site;
- Use only for the collection and transport of human biological samples for the detection of potential pathogens such as aerobic, anaerobic and facultative anaerobic bacteria, compatible with the type of device selected;
- Do not deviate from the intended use;
- Do not use the product if it is expired or the package is opened/damaged. Sterility guaranteed if unopened;
- Use the device following aseptic procedures;
- Single-use device; do not reuse. Reusing the device could contaminate the sample and/or the patient;
- Keep the device away from flames and heat sources;
- The fiber is only guaranteed to adhere to the shaft for instant sampling;
- The shaft is breakable, exert moderate pressure during sample collection;
- Store in a cool, dry place at a temperature between +2°C (35.6°F) and +30°C (86°F). Do not freeze;
- After use, the device may contain infectious microorganisms. Use appropriate PPE and dispose of the test tube and swab according to current regulations for medical waste.
- Keep out of flame or heat sources which might damage the product.
- Keep out of reach of children.
- Strictly follow the user's instructions.

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
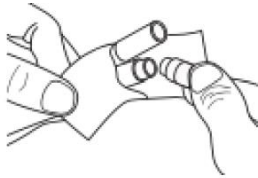
1

Open the blister pack from the "peel here" indication and remove the cap of the test tube containing the transport medium;

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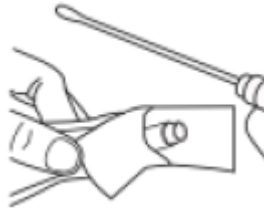2

Remove the applicator from the blister pack;

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3

Collect the sample from the concerned site;

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4

Put the applicator into the test tube containing the transport medium and seal tightly with the cap on the applicator. Write the patient's data on the label;

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Manual Processing / Procedura manual

Remove the cap with swab applicator;

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Using the swab applicator, Streak the agar plate while rolling the swab tip.

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Manual processing with swab applicator

Streak plating example

QUALITY ASSURANCE

All raw materials, components and every lot of finished product undergo strict quality inspections carried out by N.A. in order to check conformance to specifications.

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The content of the unopened and undamaged units is guaranteed to be sterile. Do not use the units in case of damage, dehydration or contamination. The Devices must be stored at +2°C (35.6°F) / +30°C (86°F) until used. Improper storage will result in a loss of efficacy. Do not use after expiration date indicated on label.

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