**Biosigma s.r.l.**

**a Dominique Dutscher Company**

Via Valletta, 6 | 30010 Cantarana di Cona (VE), Italy | Tel. ++39 0426 302224 (r.a.) |

Fax ++39 0426 302228 | SMS ++39 348 4077376 |

E-mail info@biosigmaeu.com | http://www.biosigma.com |

**M521D**

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**DISPOSABLE ENDOCERVICAL SAMPLING BRUSH NOT STERILE**

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**DIRECTIONS FOR USE**

The device is intended for the execution of endocervical canal cytology smears.

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*PHTHALATES: the material production of the devices does not contain phthalates.*

*LATEX FREE: The production material and primary and secondary packaging are latex-free*

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### PACKAGING

*The product is packaged in cellophane bags. Such a packaging ensures a 'proper waterproof protection and allows the product to reach the final consumer in safe hygienic conditions.*

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### STORAGE

*Store in a dry place. Do not expose to direct sunlight or heat sources.*

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### INSTRUCTIONS FOR USE

*Carry out the sampling by inserting the device gently in the endocervix, for the complete length of the brush head.*

*Rotate the bristles by 360° in one manoeuvre only and then take it out.*

*Deposit the sampled material on a slide, by rotating and pushing gently the brush head on the slide.*

*Fix immediately by a cytological fixing spray and then dispose of the device.*

*Caution: in case of pregnancy use it gently. Do not use it for endometrial sampling.*

### WARNINGS :

- *The device must be used by physicians or by trained staff only to carry out endocervical samplings.*

- *The product is disposable , therefore do not re-use it. Dispose after use.*

### DISCHARGE AFTER USE

*The product is disposable and therefore not re-usable. It can be discharged or burnt with other garbage in accordance with the local authority rules. Burning does not cause any toxic residues.*