**Biosigma s.r.l.**

**a Dominique Dutscher Company**

Via Valletta, 6 | 30010 Cantarana di Cona (VE), Italy | Tel. ++39 0426 302224 (r.a.) |

Fax ++39 0426 302228 | SMS ++39 348 4077376 |

E-mail info@biosigmaeu.com | http://www.biosigma.com |

**M521D**

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**STERILE PAP-TEST CERVICAL SPATULA IN PLASTIC**

**Ayre spatula perfectly suited for sampling exocervical (outside of the cervix) for precocious diagnosis of uterus cancer. Sterile individually wrapped. Latex free device**

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*This product is intended as "MEDICAL DEVICE" Class I sterile - temporarily invasive - to be used in gynaecology for the drawing of exocervical cells and on posterior fornix during screening procedures in the occurrence of cervix cancer. For professional use only.*

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### INSTRUCTIONS OF USE

1. Sampling exocervical is done first to avoid being contaminated by bleeding that can follow the endocervical sampling
2. For the drawing on the ectocervix use the Ayre spatula, by inserting the more elongated part of the spatula, in the top orifice cervical and rotating the spatula clockwise to 360 degrees
3. Smear exocervical: crawling both sides of the spatula on one side of the slide, taking care not to overlap the material, making a slight pressure to maintain the integrity of the cells and prevent degenerative processes. The material crawled must be thin because the presence of superimposed layers prevents an adequate reading of the preparation.
4. Fixing: fix the prepared immediately to prevent spontaneous cell degeneration consequent drying: even a short delay may cause damage to the cells.
5. If you are using a fixative spray form, the device must be kept at a distance of 15/20 cm from the slide: in fact, if carried out in the shortest distance, the power delivery of the cells move to the periphery and induce the formation of air bubbles in the center of the slide.
6. Allow the slide to dry in a horizontal position.
7. Dispose the Ayre spatula according to current regulations.

**WARNING:** the use of the device in case of pregnancy must be extremely delicate. Do not use for endometrial drawing. For professional use only.